# beurer

## BG 17

**READ THIS MANUAL COMPLETELY AND  
CAREFULLY BEFORE USING THIS PRODUCT**

Keep this manual in a safe location for future reference

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## 1. Important safety notes

### Signs and symbols

The following signs appear in the Safety Section (page 2, 3, 4) and in this manual on pages 6, 7, 10, 12.

**READ THIS ENTIRE MANUAL INCLUDING THE SAFETY SECTION AND ALL INSTRUCTIONS AND WARNINGS COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT. FOLLOW ALL SAFETY INSTRUCTIONS AND WARNINGS TO AVOID HAZARDOUS SITUATIONS AND TO MAKE CORRECT USE OF THIS PRODUCT.**

	<b>WARNING</b>	WARNING indicates a hazardous situation which, if not avoided, could result in death or serious injury.
	<b>CAUTION</b>	CAUTION indicates a hazardous situation which, if not avoided, may result in minor or moderate injury.
	<b>NOTICE</b>	NOTICE addresses practices not related to personal injury, such as product and/or property damage.
	Type B applied part	
	Disposal	Observe the local regulations for material disposal. Dispose of the device in accordance with local regulations. If you have any questions, please contact the local authorities responsible for waste disposal.

### CLASSIFICATION

- a. Internally powered equipment;
- b. Type B applied part;
- c. IPX0;
- d. Not Category AP/APG equipment;
- e. Mode of operation: continuous operation.

### OPERATING CONDITIONS

Temperature: 50 °F to 95 °F (10 °C to 35 °C)

Humidity: 20 % to 95 % RH

#### **WARNING:**

- Never use, or allow others to use, this unit in combination with the following medical electronic devices,
  - (1) Medical electronic implants such as pacemakers,
  - (2) Electronic life support systems such as an artificial heart/lung,
  - (3) Portable electronic medical devices such as an electrocardiograph.
- This scale passes a harmless and unnoticeable small electrical current through your body when taking a measurement. This electrical current is not felt while using the scale. This unit may cause the above mentioned medical devices to malfunction.
- This product is not intended for use by pregnant women.
- This product is not intended for use by infants, toddlers, and children under 10 years of age.
- Packaging materials are a deadly hazard for children and can cause suffocation. Remove all packaging materials immediately and keep them away from children at all times.
- Swallowing batteries and/or battery fluid can be extremely dangerous. Keep the battery and scale out of the reach of children. Should a child, or any other person swallow a battery and/or battery fluid, please call 911 immediately.
- Should battery fluid leak and come into contact with your eyes or skin, immediately rinse with plenty of clean water. Please call 911 immediately.
- Batteries should not be charged or reactivated by any other means.
- Batteries should not be taken apart, thrown in the fire or short circuited.
- Do not step on the edge of the scale (e.g. when getting on or off), otherwise it may tip.

- Do not jump on the scale.
- Protect the scale against hard knocks, temperature fluctuations and heat sources which are too close (e.g. stoves, heating radiators).
- Do not drop the scale or drop any objects on it.
- Do not step or stand on the scale when your body and/or feet are wet, for example after taking a bath or shower. There is a danger of slipping.
- Do not step on the scale when its surface is wet. There is a danger of slipping.
- Do not use the unit on slippery surfaces such as wet floors. There is a danger of slipping.
- This product does not and is not intended to provide a medical diagnosis. For medical advice please contact a licensed physician. If you have an emergency please call 911 immediately.
- Consult your physician or healthcare provider before beginning a weight reduction or exercise program.
- The measurements provided are not substitutes for physician assessments. For medical advice please contact a licensed physician.
- Please be aware that values obtained from this body analysis scale represent only an approximation of actual analytical medical data. Only a specialist physician can accurately determine body fat, body water and muscle percentage using special medical procedures (e.g. computed tomography).
- Make sure that you enter the desired unit of mass regarding the weight results.
- Always consult your physician to determine what will be suitable for you.
- This product is not a toy. Keep it out of reach of children.
- People with disabilities, such as impairments, activity limitations, and participation restrictions, or who are physically frail, should be assisted by another person when using this scale.
- Do not step on the scale if you weigh more than 330 lb (150 kg, 24 st). When in doubt contact your physician.
- Do not weigh more than one person on the scale at one time.
- Do not use this product for any purpose other than weighing human beings.
- Place the scale on level flooring, otherwise it may tip.
- If the glass platform is damaged, there could be risk of danger or injuries caused by broken glass.

### **CAUTION:**

- This unit is intended for home use only. It is not intended for professional use in hospitals or other medical facilities. This unit does not support the standards required for professional use.
- Please note that technically related measuring tolerances are possible, as the scale is not calibrated for professional medical applications.
- If a battery has leaked do not touch the battery fluid. Avoid skin contact (e.g. put on protective gloves) and clean the battery compartment with a dry cloth.
- Carefully step on the scale surface. Pay attention to avoid stepping with your toes under the scale surface. This might lead to injuries.
- Do not disassemble this unit. You could get injured.
- Do not put your fingers in any slot, inside or under the scale.

### **NOTICE:**

- Protect the scale against hard knocks, moisture, dust, chemicals, major temperature fluctuations and heat sources which are too close (e. g. stoves, heating radiators). This may damage the unit. Please see section 11 "Cleaning, maintenance and disposal" and section 12 "Technical specifications" for details.
- Do not use any aggressive solvents, cleaning agents, detergents or any other strong chemicals to clean the device.
- Never immerse and or spill water or any other liquid onto the scale, otherwise liquid will enter the scale and cause damage.
- Do not wash the scale in a washing machine, dish washer or tumble dryer.
- Always use the size and type of batteries indicated.
- Never use rechargeable batteries. This may damage the unit.
- Replace weak battery before it discharges completely. When "Lo" appears in the display please change the battery.

## **NOTICE:**

- Leaking batteries may damage the device. If you do not intend to use the scale for longer periods, remove the battery from the battery compartment before placing the device in storage.
- To avoid draining the batteries do not place any items on the scale (Please make sure that there is no weight on the feet of the scale) when not in use.
- Batteries can contain toxins that are harmful to the environment. Always dispose of batteries in accordance with applicable local regulations. Do not dispose of batteries with normal household waste.
- Observe the local regulations for material disposal. Dispose of the device in accordance with local regulations.
- Do not step on the scale if you weigh more than 330 lb (150 kg, 24 st). This may damage the unit. When in doubt contact your physician.
- Do not weigh more than one person on the scale at one time. This may damage the unit.
- Do not use mobile telephones, microwaves, and other devices which generate strong electrical or electromagnetic fields near the unit. This may result in incorrect operation of the unit.
- Do not press the buttons with excessive force or with any pointed objects.
- Never attempt to repair the instrument or adjust it yourself. This may damage the unit and impair the functions. If you need to have the unit repaired, please contact our customer service. Please see warranty for service contact. Before submitting any complaint, first check the batteries and replace them if necessary.
- Changes or modifications to the device will nullify the user warranty.

## **Content**

<b>1. Important safety notes</b> .....	2	<b>7. Limitations</b> .....	9
<b>▲ WARNING</b> .....	2	<b>8. Evaluation of results</b> .....	10
<b>▲ CAUTION</b> .....	3	Body fat .....	10
<b>NOTICE</b> .....	3	Body water .....	10
<b>2. Package Contents</b> .....	5	Muscle percentage .....	10
<b>3. Device description</b> .....	5	<b>9. Results in relation to time</b> .....	11
Overview.....	5	<b>10. Troubleshooting</b> .....	11
Measuring units.....	5	<b>11. Cleaning, maintenance and disposal</b> .....	12
General Notes.....	5	Cleaning and maintaining .....	12
<b>4. The measuring principle</b> .....	5	Disposal .....	12
<b>5. Batteries</b> .....	6	<b>12. Technical specifications</b> .....	12
<b>6. Operation</b> .....	6	<b>13. Warranty</b> .....	13
Measuring weight only .....	8	<b>14. EMC Compliance Declaration</b> .....	14
Measuring weight, body fat, body water and muscle percentage .....	8		
Explanation of the 5 levels of physical activity .....	8		
Performing the measurement .....	9		
Tips for using the scale .....	9		

## **SAVE THESE INSTRUCTIONS FOR FUTURE REFERENCE**

If the instruction manual is damaged or if you no longer have the instruction manual in your possession, please contact Beurer customer service. Please see warranty for service contact.

### **Dear Customer,**

Thank you for choosing this Beurer product. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage and air.

Please read these instructions completely and carefully before using this product and keep them for later reference. Be sure to make them accessible to other users and observe the information they contain.

With best regards,  
Your Beurer Team


## 2. Package Contents

- Glass Body Analysis scale BG17
- 1 x 3 V lithium battery CR2032
- These operating instructions
- Quick reference guide

## 3. Device description

### Overview

1. Display
2. "Down" key ◀
3. "SET" key
4. "UP" key ▶
5. Conductors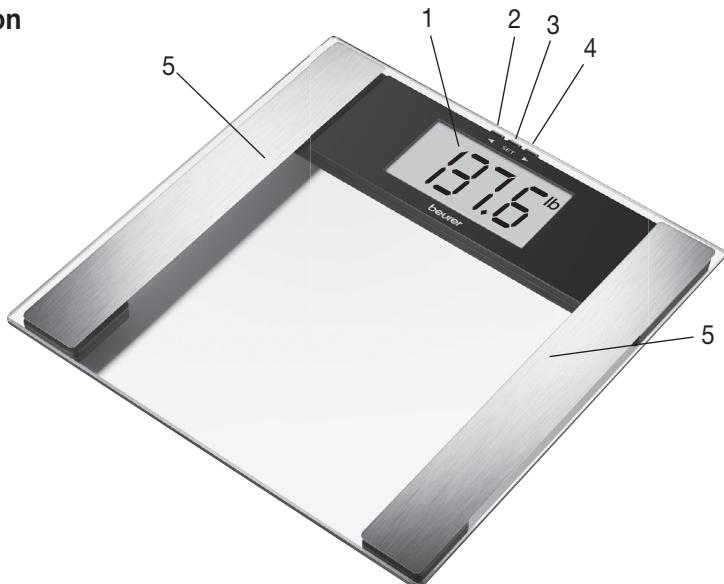

### Measuring units

lb	pound
kg	kilogram: the base unit of mass in the international System of Units.
st	stone: only for countries where British English is spoken.

### General Notes:

- The capacity of the scale is 330 lb (150 kg, 24 st).
- For weight measurement, the results are shown in 0.2 lb increments (100 g, 0.01 st).
- The measuring results of the body fat, body water and muscle percentage are shown in 0.1 % increments.
- Ages 10 to 100 and height settings from 39" to 87" (100 to 220 cm) can be preset.
- On delivery, the scale is set to "inch" and "lb". To the rear of the scale, there is a button where you can select "pound", "stone" and "kilogram" (lb, st, kg).

For product information, questions or customer service call toll free 1-800-536-0366 or email info@beurer.com

## 4. The measuring principle

This scale operates according to the B.I.A. principle (bioelectric impedance analysis). This enables the measurement of physical relationships within seconds by means of an unnoticeable and harmless signal from the conductors in the scale. The body fat percentage and other physical relationships in the body can be determined by measuring the electrical resistance (impedance) and calculating constants and individual parameters such as age, height, gender, weight and degree of physical activity.

Muscle tissue and water have good electrical conductivity, and therefore lower resistance. Fatty tissue, on the other hand, has low conductivity as fat cells hold a lower water content than muscles.

## 5. Batteries

### ⚠️ WARNING:

- Swallowing batteries and/or battery fluid can be extremely dangerous. Keep the battery and scale out of the reach of children. Should a child, or any other person swallow a battery and/or battery fluid, please call 911 immediately.
- Should battery fluid leak and come into contact with your eyes or skin, immediately rinse with plenty of clean water. Please call 911 immediately.
- Batteries should not be charged or reactivated by any other means.
- Batteries should not be taken apart, thrown in the fire or short circuited.


### ⚠️ CAUTION:

- If a battery has leaked do not touch the battery fluid. Avoid skin contact (e.g. put on protective gloves) and clean the battery compartment with a dry cloth.

### NOTICE:

- Always use the size and type of batteries indicated.
- Never use rechargeable batteries. This may damage the unit.
- Replace weak battery before it discharges completely. When “Lo” appears in the display please change the battery.
- Leaking batteries may damage the device. If you do not intend to use the scale for longer periods, remove the battery from the battery compartment before placing the device in storage.
- To avoid draining the batteries do not place any items on the scale (Please make sure that there is no weight on the feet of the scale) when not in use.
- Batteries can contain toxins that are harmful to health and the environment. Always dispose of batteries in accordance with applicable local regulations. Do not dispose of batteries with normal household waste.

If present, remove the battery insulating strip or remove the battery's protective film and insert the battery according to the polarity. If the scale fails to operate, remove the battery completely and insert it again. Replace the battery cover carefully. Your scale is equipped with a "replace battery" indicator. When the scale is operated with a battery that is too weak, "Lo" appears in the display field and the scale automatically switches off. In this case the battery must be replaced (1 x 3 V lithium battery CR2032).

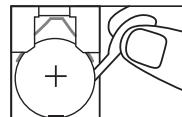New batteries will last for approximately two years if the unit is used three times per day. Trial batteries supplied with the unit may have a shorter life. Remove the batteries if you do not intend to use the unit for a period of three months or longer before placing the device in storage.

## 6. Operation

### ⚠️ WARNING!

- Never use, or allow others to use, this unit in combination with the following medical electronic devices,
  - (1) Medical electronic implants such as pacemakers,
  - (2) Electronic life support systems such as an artificial heart/lung,
  - (3) Portable electronic medical devices such as an electrocardiograph.

This scale passes a harmless and unnoticeable small electrical current through your body when taking a measurement. This weak electrical current is not felt while using the scale.

This unit may cause the above mentioned medical devices to malfunction.

- This product is not intended for use by pregnant women.
- This product is not intended for use by infants, toddlers, and children under 10 years of age.
- Packaging materials are a deadly hazard for children and can cause suffocation. Remove all packaging materials immediately and keep them away from children at all times
- Do not step on the edge of the scale (e.g. when getting on or off), otherwise it may tip.
- Do not jump on the scale.
- Protect the scale against hard knocks, temperature fluctuations and heat sources which are too close (e.g. stoves, heating radiators).

- Do not drop the scale or drop any objects on it.
- Do not step or stand on the scale when your body and/or feet are wet, for example after taking a bath or shower. There is a danger of slipping.
- Do not step on the scale when its surface is wet. There is a danger of slipping slipping.
- Do not use the unit on slippery surfaces such as wet floors. There is a danger of slipping slipping.
- This product does not and is not intended to provide a medical diagnosis. For medical advice please contact a licensed physician. If you have an emergency please call 911 immediately.
- Consult your physician or healthcare provider before beginning a weight reduction or exercise program.
- The measurements provided are not substitutes for physician assessments. For medical advice please contact a licensed physician.
- Please be aware that values obtained from this body analysis scale represent only an approximation of actual analytical medical data. Only a specialist physician can accurately determine body fat, body water and muscle percentage using special medical procedures (e.g. computed tomography).
- Make sure that you enter the desired unit of mass regarding the weight results.
- Always consult your physician to determine what will be suitable for you.
- This product is not a toy. Keep it out of reach of children.
- People with disabilities, such as impairments, activity limitations, and participation restrictions, or who are physically frail, should be assisted by another person when using this scale.
- Do not step on the scale if you weigh more than 330 lb (150 kg, 24 st). When in doubt contact your physician.
- Do not weigh more than one person on the scale at one time.
- Do not use this product for any purpose other than weighing human beings.
- Place the scale on level flooring, otherwise it may tip.
- If the glass platform is damaged, there could be risk of danger or injuries caused by broken glass.

### **CAUTION:**

- This unit is intended for home use only. It is not intended for professional use in hospitals or other medical facilities. This unit does not support the standards required for professional use.
- Please note that technically related measuring tolerances are possible, as the scale is not calibrated for professional medical applications.
- Carefully step on the scale surface. Pay attention to avoid stepping with your toes under the scale surface. This might lead to injuries.
- If a battery has leaked do not touch the battery fluid. Avoid skin contact (e.g. put on protective gloves) and clean the battery compartment with a dry cloth.
- Do not disassemble this unit. You could get injured.
- Do not put your fingers in any slot, inside or under the scale.

### **NOTICE:**

- Do not step on the scale if you weigh more than 330 lb (150 kg, 24 st). This may damage the unit. When in doubt contact your physician.
- Do not weigh more than one person on the scale at one time. This may damage the unit.
- Do not use mobile telephones, microwaves, and other devices which generate strong electrical or electromagnetic fields near the unit. This may result in incorrect operation of the unit.
- Do not press the buttons with excessive force or with any pointed objects.

## Measuring weight only

### Tap-On

Tap briefly on the weighing area of your scale.

The entire display is shown (Fig. 1) until “0.0” (Fig. 2) is displayed.

Stand on the scale without moving and distribute your weight evenly on both legs.

The scale immediately begins to measure your weight. Your measured weight is displayed shortly after this (Fig. 3). The scale switches off a few seconds after you step off of it.

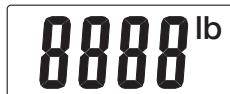
Fig. 1

Fig. 2

Fig. 3

### Instant-On

#### Note:

Alternatively, you can switch on the scale by using Instant-On.

Step carefully onto the scale, distributing your weight evenly on both legs.

The scale immediately begins to measure your weight.

After the weight is displayed the scale will automatically switch off after several seconds.

## Measuring weight, body fat, body water and muscle percentage

First your personal parameters must be stored;

- Briefly tap the scale first and wait until the “0.0” (Fig. 2) is displayed. Then press “SET”. Now the first memory preset flashes in the display (Fig. 4). Select the desired preset by pressing the buttons ▶ “up” or ◀ “down” and confirm the desired preset with the “SET” button.
- The following steps will allow you to select your desired settings with the “up” or “down” buttons and confirm with “SET”.
  - The height in inch or cm (Fig. 5).
  - The age (“Age”) in years (Fig. 6).
  - The gender setting for “Woman” or “Man” (Fig. 7).
  - The setting for individual levels of physical activity and fitness (Fig. 8).
  - The set values are displayed again consecutively.

Fig. 4

Fig. 5

Fig. 6

Fig. 7

Fig. 8

## Explanation of the 5 levels of physical activity:

Degree of activity	Physical activity
1	None.
2	Low: A small amount of light physical effort (e.g. short walks, light garden work, light exercise).
3	Medium: Physical effort for 30 minutes at least 2 to 4 times a week.
4	High: Physical effort for 30 minutes at least 4 to 6 times a week.
5	Very high: Intensive physical effort, intensive training or hard physical work for at least one hour daily.

## Performing the measurement:

### Note:

The measurement for body fat, body water and muscle percentage may only be carried out when using Tap-On.

- Tap briefly on the weighing area of your scale.
- Select the memory preset where your basic personal data are stored by pressing the “up” or “down” buttons repeatedly. The data is displayed consecutively until “0.0” (Fig. 2) appears.
- Step onto the scale barefoot and make sure you’re standing on both conductors. First your body weight is calculated and displayed.
- Remain on the scale and avoid moving; your body fat is being analyzed. This may take a few seconds.

### The following data will be displayed:

- Body fat percentage BF in % (Fig. 9)
- Water percentage  $\approx$  in % (Fig. 10)
- Muscle percentage  $\text{■■}$  in % (Fig. 11)
- Weight (lb, kg or st)
- Now all measured values are displayed consecutively and the scale switches off.

Fig. 9

Fig. 10

Fig. 11

### Tips for using the scale

- Place the scale on a hard, plane level floor (no carpet); a firm floor covering is required for correct measurement.
- If possible, always weigh yourself at the same time of day (preferably in the morning) striving for comparable conditions such as after using the toilet, on an empty stomach and unclothed, in order to achieve comparable results.
- Stand still and upright during the measurement.
- Wait a few hours after unaccustomed physical activity.
- Wait approx. 15 minutes after getting up in the morning to allow the water in the body to distribute.
- Remember that only long-term trends are important. Short-term changes in weight within a few days are usually caused by a loss of fluid. Body water plays an important role towards our general well-being.

Important when measuring body fat/body water/muscle percentage:

- The measurement may only be carried out while barefoot. Dry soles or callused skin can result in unsatisfactory results, as they have insufficient conductivity.
- Your feet, legs, calves and thighs must not touch each other, as otherwise the measurement cannot be performed correctly.

## 7. Limitations

When measuring body fat and other values, deviating and implausible results may occur in:

- growing children under 10,
- competitive athletes and body builders,
- post menopausal women,
- people with fever, undergoing dialysis treatment or with symptoms of edema or osteoporosis,
- people taking cardiovascular medication (affecting the heart and vascular system),
- people taking vascular dilating or vascular constricting medication,
- people with considerable anatomic deviations of the legs in relation to total height (length of the legs considerably shortened or lengthened),
- people with prosthetic or one leg,

- people who suffer from swelling,
- elderly people,
- people who drink coffee or alcohol.

## 8. Evaluation of results

### WARNING:

- This product does not and is not intended to provide a medical diagnosis. For medical advice please contact a licensed physician. If you have an emergency please call 911 immediately.
- Consult your physician or healthcare provider before beginning a weight reduction or exercise program.
- The measurements provided are not substitutes for physician assessments. For medical advice please contact a licensed physician.
- Please be aware that values obtained from this body analysis scale represent only an approximation of actual analytical medical data. Only a specialist physician can accurately determine body fat, body water and muscle percentage using special medical procedures (e.g. computed tomography).
- Make sure that you enter the desired unit of mass regarding the weight results.
- Always consult your physician to determine what will be suitable for you.

### Body fat

The following body fat percentages are guide values (contact your physician for further information).

A lower value is often found in athletes. Depending on the type of sports, training intensity and physical constitution, values may result which are below the recommended values stated. It should, however, be noted that there could be a danger to health in the case of extremely low values.

#### Women

Age	Very good	Good	Average	Poor
<19	<17%	17–22%	22.1–27%	>27.1%
20–29	<18%	18–23%	23.1–28%	>28.1%
30–39	<19%	19–24%	24.1–29%	>29.1%
40–49	<20%	20–25%	25.1–30%	>30.1%
>50	<21%	21–26%	26.1–31%	>31.1%

#### Men

Age	Very good	Good	Average	Poor
<19	<12%	12–17%	17.1–22%	>22.1%
20–29	<13%	13–18%	18.1–23%	>23.1%
30–39	<14%	14–19%	19.1–24%	>24.1%
40–49	<15%	15–20%	20.1–25%	>25.1%
>50	<16%	16–21%	21.1–26%	>26.1%

based on: "Principles + Labs for Physical Fitness and Wellness. 1st edition, Copyright 1999"

### Body water

According to the World Health Organization (WHO 2001), the **percentage of body water** should normally lie within the following ranges:

Women: 50–55 %

Men: 60–65 %

Children: 65–75 %

Body fat contains relatively little water. Therefore people with a high body fat percentage have body water percentages below the recommended values. With endurance athletes, however, the recommended values could be exceeded due to low fat percentages and high muscle percentage.

This scale is unsuitable for measuring body water in order to draw medical conclusions concerning age-related water retention, for example. If necessary ask your physician. Basically, a high body water percentage should be the aim.

### Muscle percentage

As the **percentage of muscle** differs from individual to individual, there are no generally valid guidelines.

## 9. Results in relation to time

### Note:

Remember that only long-term trends are important. Short-term fluctuations in weight over a few days are usually the result of fluid change, due to fluid intake or loss.

The interpretation of the results will depend on changes in your overall weight and body fat, body water and muscle percentages, as well as on the period during which these changes take place. Sudden changes within days must be distinguished from medium term changes (over weeks) and long term changes (months).

A basic rule is that short term changes in weight almost exclusively represent changes in water content, whereas medium and long term changes may also involve the fat and muscle percentages.

- If your weight reduces over the short term, but your body fat percentage increases or remains the same, you have merely lost water – e.g. after a training session, sauna session or a diet restricted only to rapid weight loss.
- If your weight increases over the medium term and the body fat percentage falls or stays the same, then you could have built up valuable muscle mass.
- If your weight and body fat percentage fall simultaneously, then your diet is working – you are losing fat mass.
- Ideally you should support your diet with physical activity, fitness or power training. This way you can increase your muscle percentage over the medium term.
- Body fat, body water or muscle percentages should not be added (certain elements of muscle tissue also contain body water).

## 10. Troubleshooting

Following error messages can appear.

Error display	Possible causes of errors	Correction of errors
“FFFF”	The maximum capacity of 330 lb (150 kg, 24 st) has been exceeded.	Weigh only the maximum permissible weight.
“Err”	The electrical resistance between the conductors and the soles of the feet is too high (e.g. in the case of thick calluses or too dry soles).	Repeat the measurement barefoot. If necessary, remove the calluses from the soles of your feet.
“Err”	The fat percentage is outside the measurable range (less than 5 % or greater than 50 %).	Repeat the measurement barefoot. If necessary, remove the calluses from the soles of your feet.
“Err”	The water percentage is outside the measurable range (less than 25 % or greater than 75 %).	Repeat the measurement barefoot. If necessary, remove the calluses from the soles of your feet.
“Err”	The muscle percentage is outside the measurable range (age and gender related).	Repeat the measurement barefoot. If necessary, remove the calluses from the soles of your feet.
“Lo”	Low Power.	Change the battery.
	No display.	Battery is empty, battery was inserted incorrectly or the scale is broken.
	Scale shows unrealistic results.	Wait until scale switches off automatically. Repeat measurement. Place the scale on level flooring (no carpet). Don't move during weighing.

## 11. Cleaning, maintenance and disposal

### NOTICE:

- Protect the scale against hard knocks, moisture, dust, chemicals, major temperature fluctuations and heat sources which are too close (e.g. stoves, heating radiators). This may damage the unit. Please see section 11 "Cleaning, maintenance and disposal" and section 12 "Technical specifications" for details.
- Do not use any aggressive solvents, cleaning agents, detergents or any other strong chemicals to clean the device.
- Never immerse and or spill water or any other liquid onto the scale, otherwise liquid will enter the scale and cause damage.
- Do not wash the scale in a washing machine, dish washer or tumble dryer.
- Observe the local regulations for material disposal. Dispose of the device in accordance with local regulations.
- Do not press the buttons with excessive force or with any pointed objects.
- Never attempt to repair the instrument or adjust it yourself. This may damage the unit and impair the functions. If you need to have the unit repaired, please contact our customer service. Please see warranty for service contact. Before submitting any complaint, first check the batteries and replace them if necessary.
- Changes or modifications to the device will nullify the user warranty.

### Cleaning and maintaining

To keep your scale in the best condition and protect the unit from damage follow the directions below:  
The scale should occasionally be cleaned with a damp cloth.

Do not use aggressive cleaning agents and never submerge the scale. The unit is not waterproof. Do not immerse the unit in water and do not spill water over the unit. If your scale accidentally gets wet immediately clean off the water.

Store the unit in a safe and dry location. Do not expose the unit to direct sunlight, extreme hot or cold temperatures or humidity. Thoroughly dry any moisture off the unit before storing. Store the unit safely out of the reach of children. Do not subject the unit to strong shocks, such as dropping the unit on the floor. Always use the unit following the instructions provided in this manual.

### Disposal

### NOTICE:

- Batteries should be disposed of separately from household waste. Always dispose of batteries as per your local regulations. Again, do not dispose of batteries with normal household waste.
- Observe the local regulations for material disposal. Dispose of the device in accordance with local regulations. If you have any questions, please contact the local authorities responsible for waste disposal.

## 12. Technical specifications

Battery type	1 x 3 V lithium battery CR2032
Measuring range	2.2 lb to 330 lb (1 kg to 150 kg)
Switch activation force	4.4 lb (2 kg)
Weight unit	0.2 lb increments (100 g)
Body fat, water, muscle percentage unit	0.1 % increments
Electrical current of B.I.A.	max. 5 µA / 1 V
Fat percentage range	5 % to 50 %
Water percentage range	25 % to 75 %
Muscle percentage range	25 % to 80 %
Temperature operating range	41 °F to 95 °F (5 °C to 35 °C)
Storage temperature range	-4 °F to 140 °F (-20 °C to 60 °C)
Scale size	11.8" x 11.8" x 0.8" (300 mm x 300 mm x 20 mm)
Net weight of scale	3.5 lb (1.6 kg)

Subject to technical modifications.

## **13. Warranty**

### **Limited Lifetime Warranty For Original Purchaser**

Your Beurer Glass Body Analysis Scale, Model BG17, excluding the battery, is warranted to be free from defects in materials and workmanship for the life of the product under normal conditions of intended use and service. This warranty extends only to the original retail purchaser and does not extend to retailers or subsequent owners.

We will, at our option, repair or replace the Beurer Glass Body Analysis Scale, Model BG17, without additional charge, for any part or parts covered by these written warranties. No refunds will be given. Repair or replacement is our only responsibility and your only remedy under this written warranty. If replacement parts for defective materials are not available, Beurer reserves the right to make product substitutions in lieu of repair or replacement.

For warranty service contact our customer service department at 1-800-536-0366 or at info@beurer.com to provide a description of the problem. If the problem is deemed to be within the scope of the limited lifetime warranty, you will be asked to mail the product at your costs in its original package with proof of purchase, your name, address and phone number. If the problem is not deemed to be within the scope of the limited lifetime warranty, we will provide a quotation for repair respectively replacement and return shipping fee.

This warranty does not cover damage caused by misuse or abuse; accident; the attachment of unauthorized accessory; alteration to the product; improper installation; misapplication; lack of reasonable care with respect to the product; unauthorized repairs or modifications; improper use of electrical/power supply; old worn battery; normal wear; loss of power; dropped product; malfunction or damage of an operating part as a result of failure to comply with instructions for use or to provide manufacturer's recommended maintenance; transit damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of Beurer. This warranty is void if the product is ever used in a commercial or business environment. The maximum liability of Beurer under this warranty is limited to the purchase price actually paid by the customer for the product covered by the warranty, as confirmed by proof of purchase, regardless of the amount of any other direct or indirect damage suffered by the customer.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adaptation to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

**THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. ANY IMPLIED WARRANTIES, OBLIGATIONS, OR LIABILITIES, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTY OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED IN DURATION TO THE DURATION OF THIS APPLICABLE WRITTEN WARRANTY.** Some states do not allow limitations on how long an implied warranty lasts, so the above limitations may not apply to you.

**IN NO EVENT SHALL BEURER BE LIABLE FOR ANY SPECIAL, INCIDENTAL, INDIRECT OR CONSEQUENTIAL DAMAGES FOR BREACH OF THIS OR ANY OTHER WARRANTY, EXPRESS, IMPLIED OR ANY OTHER THEORY OF LIABILITY, WHATSOEVER.** Some states do not allow the exclusion or limitation of special, incidental, or consequential damages, so the above limitation may not apply to you.

Beurer does not authorize anyone, including, but not limited to, retailers, the subsequent consumer purchaser of the product from a retailer or remote purchaser, to obligate Beurer in any way beyond the terms set forth herein.

This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resealed products, including but not limited to sale of such products on Internet auction sites and/or products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate in connection with any products or parts thereof which are repaired, replaced, altered, or modified, without the prior explicitly written consent of Beurer.

This warranty gives you specific legal rights, and you may also have other rights which may vary from state to state.

For more information regarding our product line in the USA, please visit: [www.beurer.com](http://www.beurer.com)

Distributed by:

Beurer North America LP  
Hallandale Beach, FL 33009  
[www.beurer.com](http://www.beurer.com)

Questions or comments? Call toll free 1-800-536-0366 or contact [info@beurer.com](mailto:info@beurer.com)

Made in China.

## 14. EMC Compliance Declaration

### ELECTROMAGNETIC COMPATIBILITY INFORMATION

Table 1

For all ME EQUIPMENT and ME SYSTEMS

Guidance and manufacturer's declaration – electromagnetic emissions		
The BG 17 is intended for use in the electromagnetic environment specified below.		
The customer or the user of the BG 17 should assure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The BG 17 uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	

Table 2

For all ME EQUIPMENT and ME SYSTEMS

Guidance and manufacturer's declaration – electromagnetic immunity			
The BG 17 is intended for use in the electromagnetic environment specified below. The customer or the user of the BG 17 should assure that it is used in such an environment.			
IMMUNITY test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 6 kV contact ± 8 kV air	± 6 kV contact ± 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

**Table 3****For ME EQUIPMENT and ME SYSTEMS that are not LIFE-SUPPORTING**

<b>Guidance and manufacturer's declaration – electromagnetic immunity</b>			
The BG 17 is intended for use in the electromagnetic environment specified below. The customer or the user of the BG 17 should assure that it is used in such an environment.			
<b>IMMUNITY test</b>	<b>IEC 60601 test level</b>	<b>Compliance level</b>	<b>Electromagnetic environment – guidance</b>
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	[E <sub>1</sub> ] V/m 3 V/m	<p>Portable and mobile RF communications equipment should be used no closer to any part of the BG 17, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p><b>Recommended separation distance</b></p> $d = \frac{[3.5]}{[E_1]} \sqrt{P} \quad 80 \text{ MHz to } 800 \text{ MHz}$ $d = \frac{[7]}{[E_1]} \sqrt{P} \quad 800 \text{ MHz to } 2.5 \text{ GHz}$ <p>Where <math>P</math> is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and <math>d</math> is the recommended separation distance in meters (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey,<sup>a</sup> should be less than the compliance level in each frequency range.<sup>b</sup></p> <p>Interference may occur in the vicinity of equipment marked with the following symbol: </p>
<p><b>NOTE 1</b> At 80 MHz and 800 MHz, the higher frequency range applies.</p> <p><b>NOTE 2</b> These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.</p>			
<p>a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the BG 17 is used exceeds the applicable RF compliance level above, the BG 17 should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the BG 17.</p> <p>b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than [V1] V/m.</p>			

**Table 4****For ME EQUIPMENT and ME SYSTEMS that are not LIFE-SUPPORTING**

<b>Recommended separation distances between portable and mobile RF communications equipment and the BG 17</b>		
Rated maximum output power of transmitter  W	<b>Separation distance according to frequency of transmitter m</b>	
	<b>80 MHz to 800 MHz</b>	<b>800 MHz to 2.5 GHz</b>
	$d = \left[ \frac{3,5}{E_1} \right] \sqrt{P}$	$d = \left[ \frac{7}{E_1} \right] \sqrt{P}$
0.01	0.117	0.233
0.1	0.368	0.737
1	1.167	2.333
10	3.689	7.37
100	11.667	23.333

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be determined using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

**NOTE 1** At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

**NOTE 2** These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

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Fuente: „Principles + Labs for Physical Fitness and Wellness. 1st edition, Copyright 1999“