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Contents

1. Getting to know your instrument.....	11
1.1 What is the bottom toning device?.....	11
1.2 What is EMS and what can it do?	11
1.3 Included in delivery.....	12
2. Important notes	12
3. Initial use	15
3.1 Inserting the battery.....	15
3.2 Attaching the gel film	15
3.3 Usage.....	15
4. Replacing the gel film	16
5. Disposal.....	16
6. Technical specifications	16
7. What if there are problems?	17

Dear Customer,

We are very pleased that you have decided in favor of a product from our range. Our name stands for high quality and thoroughly tested products in the fields of heat, gentle therapy, blood pressure/diagnosis, weight, massage and air. Please read these instructions for use carefully and comply with the directions.

With best regards
Your BEURER Team

	LOT number
	Expiry date
	Manufacturer
	2 x AAA battery
	Caution! Read the instructions for use.
	Follow instructions for use
	Application part, type BF

1. Getting to know your instrument

1.1 What is the glute toning device?

The glute toning device is a self-adhesive EMS device for muscle training and relaxation. The device can be applied with accuracy thanks to the use of EMS technology. Made from medical-quality silicone rubber, the elegantly designed EMS pad is extremely slim and flexible and adapts perfectly to the area to be treated. The high-tech circuit provides energy-efficient treatments of 20 minutes each.

1.2 What is EMS and what can it do?

Electrical muscle stimulation (EMS) is a widespread and generally recognised method and has been used in sports medicine and rehabilitation for years. In sports and fitness, EMS is used to complement conventional muscle

training, to increase the performance of muscle groups and to adjust physical proportions to achieve the desired aesthetic results. There are two different types of EMS application: one is for targeted strengthening of the muscles (activating application), and the other is to achieve a relaxing, restful effect (relaxing application). EMS devices work by passing electrical currents over the skin. The gel film is used as a transfer medium and is subject to natural wear and tear. The gel film must be replaced if it stops providing sufficient contact, as this will prevent the EMS pad from sticking to the skin. If it is not replaced, the partially increased current density could irritate the skin.

1.3 Included in delivery

- 1 x glute toning device
- 4 x self-adhesive gel film
- 2 x 1.5 V AAA battery

Replacement items

- 1 x self-adhesive gel film set, item no.: 647.14

When necessary, please contact Customer Services at the address provided in good time.

IMPORTANT!

Only use the glute toning device:

- On people.
- For the intended purpose and as specified in these instructions for use. Any improper use can be dangerous.
- For external use.
- Use only with the original accessories supplied, which can be re-ordered. Failure to do so invalidates the warranty.

2. Important notes

The device is not a substitute for medical consultation and treatment. Consult your doctor first if you are experiencing any pain or are suffering from an illness.

WARNING!

To avoid damage to health, we strongly advise against using the glute toning device in the following situations:

- If you have implanted electrical devices (e.g. a pacemaker).
- If you have metal implants.
- If you use an insulin pump.
- If you have a high fever (e.g. $> 39^{\circ}\text{C}$).
- If you have a known or acute cardiac arrhythmia, or disorders of the heart's impulse and conduction system.
- If you suffer from a seizure disorder (e.g. epilepsy).
- If you are pregnant.
- If you have cancer.
- After an operation, if strong muscle contractions could affect the healing process.
- If you have had a back operation in the past and the back pain is due to the surgery or an accident.
- If you suffer from metabolic arthritis.
- The device must never be used near to the heart. The stimulation electrodes must not be placed on any part of the front ribcage (where the ribs and breastbone are located), especially not on the two large pectorals. This can increase the risk of ventricular fibrillation and induce cardiac arrest.

- On the skeletal skull structure, or around the mouth, throat or larynx.
- In the neck area/carotid artery area/above the neck.
- In the genital area.
- On acutely or chronically diseased (injured or irritated) skin (e.g. inflamed skin - whether painful or not, reddened skin, rashes, e.g. allergies, burns, bruises, swellings, both open and healing wounds, and post-operative scars where the healing process could be affected).
- In humid environments (e.g. in the bathroom) or when bathing or showering.
- Do not use after consuming alcohol.
- Do not use if connected to a high-frequency surgical device.

Before using the device, consult your doctor if any of the following applies to you:

- Serious illnesses, in particular if you suspect or have been diagnosed with high blood pressure, a blood coagulation disorder, propensity to thrombo-embolic conditions or recurrent malignant growths.
- Skin diseases or open wounds.
- Unexplained chronic pain in any part of the body.
- Diabetes.
- Any sensory impairment that reduces the feeling of pain (e.g. metabolic disorders).
- If you are receiving medical treatment.
- If you are already having stimulation treatment for existing conditions.
- If you suffer from persistently irritated skin due to long-term stimulation at the same electrode site.

 WARNING!

- Swallowing (rechargeable) batteries can be extremely dangerous. Keep the (rechargeable) batteries and items out of the reach of small children. If a (rechargeable) battery is swallowed, seek medical attention immediately.
- If a battery has leaked, avoid contact with the skin, eyes and mucous membranes. If applicable, rinse the affected areas immediately with water and consult a doctor without delay.
- Use identical or equivalent battery types only (see 6. "Technical data").
- Do not charge, disassemble, short-circuit or throw the batteries into an open fire.
- Protect the batteries from excessive heat.
- Remove the batteries from the item if they are empty or you do not intend to use the item for a prolonged period of time. This prevents damage which could occur when the batteries expire.
- If required, clean the contacts of the batteries and the device before inserting. Risk of overheating!

PRECAUTIONS:

- Hold the device away from sources of heat and do not use it in close proximity (approx. 1 m) to shortwave or microwave devices (e.g. mobile phones), as doing so can result in unpleasant current peaks.
- Do not expose the device to direct sunlight or high temperatures.
- Protect the device from dust, dirt and humidity. Never immerse the device in water or other liquids.

- The device is suitable for self-treatment.
- If the device does not work properly, or you feel unwell or experience pain, stop using it immediately.
- Do not adapt the glute toning device in any way, for example, by cutting or trimming it. This will produce a higher current density and can be dangerous.
- Do not use whilst asleep, driving a vehicle or operating machinery.
- Do not use whilst undertaking any activity where an unexpected reaction (e.g. strong muscle contractions even at low intensity) could be dangerous.
- Do not use during strenuous activity (e.g. sport)
- Do not use on wet skin if this reduces the adhesive capability of the glute toning device.
- Ensure that no metallic objects (e.g. belt buckles) come into contact with the electrodes during stimulation. If you are wearing jewellery or have piercings in the area to be treated, these must be removed before using the device. Failure to do so could result in spot burns.
- Keep the device away from children.
- Pre-treating the skin with greasy creams or lotions is not recommended.
- Do not use adhesive tapes, bandages or any aid other than the designated gel film to apply the glute toning device to your skin. Otherwise the unequal distribution of the gel film could lead to skin lesions.
- During the initial few minutes, use the device while sitting or lying down to minimise the risk of accidental injuries as a consequence of isolated cases of vagal responses (feeling of faintness). If you feel faint, switch off the device immediately, lie down and support the legs in an elevated position (approx. 5 – 10 min).

Damage

- If the glute toning device is damaged or you have any doubts, do not use it and contact your retailer or Customer Services at the address provided.
- Switch the device off immediately if it is faulty or not working properly.
- Do not attempt to open and/or repair the glute toning device yourself. Repairs may only be carried out by Customer Services or authorised retailers. Failure to comply will result in voiding of the warranty.
- The manufacturer is not liable for damage resulting from improper or careless use.

Cleaning

- Remove the batteries from the device before cleaning.
- Clean the device after use with a soft, slightly damp cloth. If they are very dirty, you can also moisten the cloth with a mild soapy solution.
- Ensure that no water enters the device. If this should happen, immediately remove the battery and only use the device again when it has fully dried out.
- Do not use any chemical or abrasive cleaning agents.

Storage

- Re-apply the protective film to the gel film after use.
- If you do not plan to use the device for a long period of time, we recommend that you store it in the original packing in a dry environment, ensuring it is not weighed down by additional items.

3. Initial use

3.1 Inserting the battery

1. Open the battery compartment lid by sliding the battery compartment lid upwards [1].
2. Insert the 2 x 1.5 V AAA batteries. Make sure that the batteries are inserted correctly [2].
3. Carefully close the battery compartment lid.

3.2 Attaching the gel film

1. Carefully remove the blue protective film [3].
2. Carefully attach the gel film to the glute toning device and detach the protective film [4]. Ensure that the edge of the gel film does not protrude over the glute toning device. Gel film that has been applied slightly askew will not affect the function.

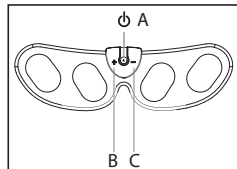
3.3 Usage

- Place the glute toning device on the part of the body to be treated.

1. Press and hold down the on/off ϕ button [A] for 1.5 seconds until the device switches on. You will hear a beep.
2. Use the +/- buttons to increase [B] or reduce [C] the intensity.
3. The automatic switch-off activates after 20 minutes

For optimal results we recommend a 4-week training plan. Please note the following points:

1. 1-2 days rest each week.
2. Note down the highest intensity level reached after each session and gradually increase this from time to time.
3. Higher intensity levels can achieve better results, but the stimulation should never be uncomfortable.
4. Ensure 20 minutes of stimulation for each use.

WARNING!

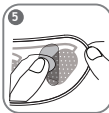
- Make sure that you attach the gel film to the device on the side where the blue protective film was, otherwise the skin could become irritated.

i Important note!

- Switch off the glute toning device before removing it.
- Replace the batteries if they are low.
- The device will only work if there is sufficient contact with the skin. If the device is not pressed on there will be no stimulation.
- Do not use the glute toning device more than once per day.
- In the event of continuous beeping, shut down the device. Clean the gel film and try to restart the program. If the continuous beeping persists, replace the gel film.
- If the adhesive capability of the gel film decreases, replace immediately. Do not use the glute toning device again until the gel film has been replaced. Otherwise the unequal distribution of the gel film could lead to skin lesions.
- In the event of unpleasant current peaks, switch off the glute toning device and clean the gel film.

4. Replacing the gel film

1. Fully remove the old, used gel film [5].
2. Carefully remove the blue protective film [3].
3. Carefully attach the gel film to the glute toning device and detach the protective film. Ensure that the edge of the gel film does not protrude over the glute toning device [4].

⚠ WARNING!

Make sure that you attach the gel film to the device on the side where the blue protective film was, otherwise the skin could become irritated.

5. Disposal

The empty, completely flat batteries should be disposed of through specially designated collection boxes, recycling points or electronics retailers. You are legally required to dispose of the batteries.

Note: The codes below are printed on batteries containing harmful substances:

Pb = Battery contains lead, Cd = Battery contains cadmium, Hg = Battery contains mercury.

Please dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.

6. Technical specifications

Type	EM 25 glute toning device
Output wave form	Biphasic rectangular pulse
Pulse length	400 μ s
Pulse frequency	4-50 Hz
Output voltage	max. 90 vpp (1000 ohm)

Output current	max. 90 mApp (1000 ohm)
Voltage supply	2 x 1.5 V AAA battery
Treatment time	20 minutes, the device then switches off automatically
Intensity	Adjustable from 0 to 15
Operating conditions	10°C–40°C (50°F–104°F) at a relative humidity of 30–85%
Storage conditions	-10°C – 50°C (14°F – 122°F) with a relative humidity of 10–95%
Dimensions	429 x 125 x 21 mm
Weight	110 g (without batteries), 133 g (with batteries)

i Note:

If the device is not used according to the instructions specified, perfect functionality cannot be guaranteed!
We reserve the right to make technical changes to improve and develop the product.

7. What if there are problems?

Problem	Possible cause/remedy
Device will not switch on.	- Press and hold down the “ⓘ” button again for 1.5 seconds. - Check that the batteries are inserted correctly and try to switch on the device again. - The batteries could be flat. Replace the batteries and try to switch on the device again.
The gel film no longer adheres to the skin.	- Replace the gel film.
The device beeps unusually during use.	- Press and hold down the “ⓘ” button for 1.5 seconds to switch off the device. Then press and hold down the “ⓘ” button for 1.5 seconds to switch on the device again. - There is insufficient skin contact with the gel pads. Replace the gel pads. - The batteries could be low and must be replaced.

You are unable to feel any stimulation.	- Press and hold down the “ϕ” button again for 1.5 seconds. - Ensure that the gel film does not protrude from the device and try again. - Gradually increase the intensity level.
The skin at the area of treatment is red.	- Stop treating this area immediately. Wait until the skin has returned to its normal colour. Please consult a dermatologist if skin irritation persists.
The intensity suddenly begins to weaken.	- Replace the batteries and try again.

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