- | | |
|--|---|
| (D) Glaswaage
Gebrauchsanweisung | (GR) Γυάλινη ζυγαρία
Οδηγίες χρήσης |
| (GB) Glass scale
Instruction for use | (DK) Glasvægt
Betjeningsvejledning |
| (F) Pèse-personne en verre
Mode d'emploi | (S) Glasvåg
Bruksanvisning |
| (E) Báscula de vidrio para personas
Manual de instrucciones | (N) Glassvekt
Bruksveiledning |
| (I) Bilancia in vetro
Istruzioni per l'uso | (FIN) Lasivaaka
Käyttöohje |
| (TR) Cam Terazi
Kullanım kılavuzu | (CZ) Skleněná váha
Návod k použití |
| (RUS) Стеклянные весы
Инструкция по применению | (SLO) Steklena tehtnica
Navodila za uporabo |
| (PL) Waga szklana
Instrukcja obsługi | (H) Üvegmérleg
Használati utasítás |
| (NL) Weegschaal met glazen
weegplateau
Gebruiksaanwijzing | (RO) Cântar de sticlă
Instrucţiuni de utilizare |
| (P) Balança de vidro
Instruções de utilização | الميزان الزجاجي
دليل الاستعمال |

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- (GB) Getting started
(F) Mise en service
(E) Puesta en marcha
(I) Messa in funzione
(TR) İlk çalıştırma

- (RUS) Ввод в эксплуатацию
(PL) Uruchomienie
(NL) Ingebruikname
(P) Colocação em funcionamento

- (GR) Έναρξη λειτουργίας
(DK) Ibrugtagning
(S) Börja använda vågen
(N) Bruk
(FIN) Käyttöönotto

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GB General advice

- Max. capacity 150 kg (330 lb / 24 st), Graduation 100 g (0.2 lb / 1 lb).
- Cleaning: You can clean the scale with a damp cloth and a little washing up liquid, if required. Never immerse the scale in water or rinse it under running water.
- Protect the scale against knocks, damp, dust, chemicals, drastic changes in temperature, elec-

tromagnetic fields and sources of heat that are too near.

- Never step onto the outer edge of the scale with one foot as this could cause the scales to tip!
- Keep packaging material away from children!
- Not intended for commercial use.
- Should you have any questions regarding operation and usage of our appliances, please contact your local retailers or Customer Service.
- Before making a claim, please check the batteries and replace them if necessary.
- Repairs must only be carried out by Customer Services or authorised suppliers.
- If your skin or eyes come into contact with battery fluid, flush out the affected areas with water and seek medical assistance.
- ⚠ **Choking hazard!** Small children may swallow and choke on batteries. Store the batteries out of the reach of small children.
- Observe the plus (+) and minus (-) polarity signs.
- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- Protect the batteries from excessive heat.
- ⚠ **Risk of explosion!** Never throw batteries into a fire.
- Do not charge or short-circuit batteries.
- If the device is not to be used for a long period, take the batteries out of the battery compartment.
- Use identical or equivalent battery types only.
- Always replace all batteries at the same time.
- Do not use rechargeable batteries.
- Do not disassemble, split or crush the batteries.
- The empty, completely flat batteries must be disposed of through specially designated collection boxes, recycling points or electronics retailers. You are legally required to dispose of the batteries.
- The codes below are printed on batteries containing harmful substances:
Pb = Battery contains lead,
Cd = Battery contains cadmium,
Hg = Battery contains mercury.
- For environmental reasons, do not dispose of the device in the household waste at the end of its useful life. Dispose of the unit at a suitable local collection or recycling point. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.

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