
D	Glaswaage Gebrauchsanleitung	GR	Γυάλινη ζυγαρία Οδηγίες χρήσης
GB	Glass scale Instruction for Use	DK	Glasvægt Betjeningsvejledning
F	Pèse-personne en verre Mode d'emploi	S	Glasvåg Bruksanvisning
E	Báscula de vidrio para personas Instrucciones para el uso	N	Glassvekt Bruksveiledning
I	Bilancia in vetro Instruzioni per l'uso	FI	Lasivaaka Käyttöohje
TR	Cam Terazi Kullanma Talimatı	CZ	Skleněná váha Návod k použití
RUS	Стеклянные весы Инструкция по применению	SLO	Steklena tehničica Navodila za uporabo
PL	Waga szklana Instrukcja obsługi	H	Üvegmérleg Használati utasítás
NL	Weegschaal met glazen weegplateau	RO	Cântar de sticla Instrucțiuni de utilizare
P	Gebruikershandleiding Balança de vidro		الميزان الزجاجي دليل الاستعمال
	Instruções de utilização		

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- GB Getting started
F Mise en service
E Puesta en marcha
I Messa in funzione
TR İlk çalıştırma

- D Batterie einlegen oder Isolierschutzstreifen entfernen.
GB Insert batteries or remove insulating strip.
F Insérez les piles ou retirez la bande de protection isolante.
E Colocar las pilas o quitar las tiras de protección de aislamiento.

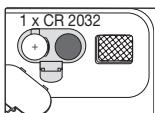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- Max. capacity 150 kg (330 lb / 24 st), Graduation 100 g (0,2 lb / 0,01 st).
 - Cleaning: You can clean the scale with a damp cloth and a little washing up liquid, if required. Never immerse the scale in water or rinse it under running water.
 - Protect the scale against knocks, damp, dust, chemicals, drastic changes in temperature, electromagnetic fields and sources of heat that are too near.
 - Stand still whilst being weighed!
 - Never step onto the outer edge of the scale with one foot as this could cause the scales to tip!
 - Keep packaging material away from children!
 - Not intended for commercial use.
 - Should you have any questions regarding operation and usage of our appliances, please contact your local retailers or Customer Service.
 - Before making a claim, please check the batteries and replace them if necessary.
 - Repairs must only be carried out by Customer Services or authorised suppliers.
 - Spent batteries and rechargeable batteries do not constitute normal household waste! They are considered to be toxic waste and, as such, should be disposed of in special containers, toxic waste collection points or brought to electrical goods dealers.
- You are legally obliged to dispose of the batteries.
- Note: Batteries containing pollutant substances are marked as follows:
Pb = Battery contains lead,
Cd = Battery contains cadmium,
Hg = Battery contains mercury.
 - Follow local regulations regarding the disposal of materials. The appliance should be disposed of according to Regulation 2002/96/EC-WEEE (Waste Electrical and Electronic Equipment). In case of queries, contact the municipal authorities responsible for waste disposal in your area.

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