# beurer

# GS 420 tara

- D** Universal-Glaswaage  
Gebrauchsanweisung
- GB** Universal glass scale  
Instruction for use
- F** Pèse-personne universel en verre  
Mode d'emploi
- E** Báscula de vidrio universal  
Manual de instrucciones
- I** Bilancia universale in vetro  
Instruzioni per l'uso
- TR** Genel amaçlı cam terazi  
Kullanım kılavuzu
- RUS** Универсальные стеклянные весы  
Инструкция по применению
- PL** Uniwersalna waga szklana  
Instrukcja obsługi
- NL** Universele glazen weegschaal  
Gebruiksaanwijzing
- P** Balança de vidro universal  
Instruções de utilização

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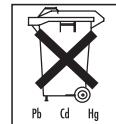
## General advice

- Max. capacity 150 kg, Graduation 50 g.
- Cleaning: You can clean the scale with a damp cloth and a little washing up liquid, if required. Never immerse the scale in water or rinse it under running water.
- Protect the scale against knocks, damp, dust, chemicals, drastic changes in temperature, electromagnetic fields and sources of heat that are too near.
- Never step onto the outer edge of the scale with one foot as this could cause the scales to tip!
- Keep packaging material away from children!
- Not intended for commercial use.

- Should you have any questions regarding operation and usage of our appliances, please contact your local retailers or Customer Service.
- Before making a claim, please check the batteries and replace them if necessary.
- Repairs must only be carried out by Customer Services or authorised suppliers.
- Spent batteries and rechargeable batteries do not constitute normal household waste! They are considered to be toxic waste and, as such, should be disposed of in special containers, toxic waste collection points or brought to electrical goods dealers.

You are legally obliged to dispose of the batteries.

- Note: Batteries containing pollutant substances are marked as follows:  
Pb = Battery contains lead,  
Cd = Battery contains cadmium,  
Hg = Battery contains mercury.
- Please dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any queries, please contact the appropriate local authorities.

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