- | | |
|--|--|
| <p>(D) Paraffinbad
Gebrauchsanweisung2-9</p> <p>(GB) Paraffin bath
Instructions for use10-16</p> <p>(F) Bain de paraffine
Mode d'emploi.....17-24</p> <p>(E) Baño de parafina
Instrucciones de uso24-31</p> <p>(I) Bagno di paraffina
Istruzioni per l'uso.....32-38</p> <p>(TR) Parafin banyosu
Kullanma talimatı39-45</p> | <p>(RUS) Парафиновая ванночка
Инструкция
по применению46-54</p> <p>(PL) Wanienka do parafiny
Instrukcja obsługi.....55-61</p> <p>(NL) Paraffinebad
Gebruiksaanwijzing.....62-68</p> <p>(P) Banho de parafina
Instruções
de utilização.....69-76</p> <p>(GR) Λουτρό παραφίνης
Οδηγίες χρήσης77-84</p> |
|--|--|

Inhalt

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Contents

1. Included in delivery	10	7. Operation.....	14
2. Safety notes	10	8. Cleaning	16
3. Getting to know your instrument	12	9. Replacement parts and wearing parts...	16
4. Intended use	12	10. Disposal.....	16
5. Unit description	13	11. Technical data	16
6. Initial use	13		

Dear Customer,

Thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage, beauty, baby and air.

Please read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

With kind regards,
Your Beurer team

1. Included in delivery

- 1 MPE 70 paraffin bath
- 1 lid
- 1 protective grille
- 2 450 g bags of paraffin wax
- 30 plastic sleeves

Warning

- The device is only intended for domestic/private use, not for commercial use.
- This device may be used by children over the age of 8 and by people with reduced physical, sensory or mental skills or a lack of experience or knowledge, provided that they are supervised or have been instructed on how to use the device safely, and are fully aware of the consequent risks of use.
- Children must not play with the device.
- Cleaning and user maintenance must not be performed by children unless supervised.
- The mains connection cable of this device cannot be replaced. The device must be scrapped in the event of damage to the cable.

2. Safety notes

Signs and symbols

The following symbols are used in these instructions for use, on the packaging and on the type plate for the device:

 Warning	Warning notice indicating a risk of injury or damage to health.
 Important	Safety note indicating possible damage to the unit/accessory.
	The CE symbol certifies that the product complies with the requirements imposed upon the manufacturer by the European Community.
	Disposal in accordance with EC Directive WEEE (Waste Electrical and Electronic Equipment).
	Manufacturer.
	The device is double protected and therefore corresponds to protection class 2.

Important safety notes – before use

Warning

- The device is intended solely for the purpose described in these instructions for use. The manufacturer is not liable for damage resulting from improper or careless use. This device is not suitable for commercial use or for use in the medical domain. Only use the device in the private domain.
- Repairs may only be conducted by authorised service centres, as special tools and/or parts are necessary for repair, otherwise the guarantee is invalidated.
- Do not use the device if you have heart disease, flushed skin, inflammations, swelling, sensitive areas, diabetes, open wounds, burns, eczema, psoriasis or reduced sensitivity to pain.
- Do not use the device if you are not sensitive to heat.
- If you are pregnant or suffer from health problems, please consult your doctor before using the device.
- If you suffer from acute inflammations or a peripheral vascular disease, please consult your doctor before using the device.
- Never operate the paraffin bath without paraffin wax.
- Remove jewellery from your hands and feet before use.
- Never heat the paraffin wax in its plastic packaging. Never heat the paraffin wax on the hob, over a naked flame or in a microwave. Only heat the paraffin wax in the paraffin bath.
- Children do not recognise the danger posed by electrical devices. Make sure that the device cannot be used by children without supervision.
- Check that the device and the mains cable are in perfect working order before each usage. In the event of damage, the device must not be used.
- Before connecting the device to the mains, check whether the local mains voltage corresponds to the voltage specified on the device.
- Do not use the device near water (e.g. bath, wash basin etc.).
- Do not use the device while bathing, showering or sleeping.
- Never place the device in water or other liquids. Store the device in a safe place.
- Never place the device on easily damaged surfaces.
- When using the device, ensure that it is placed on a stable and level surface.
- Keep the device and mains cable away from hot surfaces. Do not place it on top of other devices, as the hot paraffin wax may cause damage.

- Do not use the device in the vicinity of aerosol products or equipment that emits oxygen.
- Do not use the device outside. The device is only to be used inside.
- Before use, ensure that the mains cable does not pose a trip hazard.
- Do not use the device in bed, whilst sleeping or if you are tired.
- Never use the device under a blanket or towel etc.
- Only use the device with the supplied accessories.
- Do not overfill the paraffin bath. Ensure that the liquid paraffin wax does not exceed the MAX fill indicator line.
- If the paraffin wax has completely melted, the device may be very hot.
- Before submerging your hands, feet or elbows in the liquid paraffin wax, use the inside of your wrist to check that the temperature of the wax is comfortable.

Important safety notes – during use

Warning

- Should a fault occur during use, switch off the device immediately and remove the mains plug from the socket.
- Keep the hot paraffin wax away from your eyes and other body parts that are sensitive to heat.
- If the paraffin wax permanently feels too hot or too cold against your skin, this may be a physical problem. If this is the case, consult your doctor.
- Make sure that the device is always filled with paraffin wax up to the MIN fill indicator line during use.
- Do not apply the paraffin wax to the treatment area for longer than 20 minutes.
- Do not shake or move the device during use. You could sustain scald or burn injuries.
- If there is hot paraffin wax in the paraffin bath, do not pour in any water.
- If pain, irritation, dermatitis or discomfort occurs, terminate the treatment and consult your doctor.
- Should the device accidentally fall into water, disconnect the mains plug immediately. Have the device checked by a qualified electrician before it is used again.

Important safety notes – after use

Important

- Switch off the device after use by removing the mains plug from the socket.
- Never remove the mains plug from the socket by pulling on the cable or with damp hands.
- Do not use any sharp or pointed objects to remove the wax block from the paraffin bath. You could damage or scratch the paraffin bath.
- If you are not going to use the device for an extended period of time, store it in a cool, dry and dust-proof place.
- If you should have any questions about using our devices, please contact either your retailer or Customer Services.

3. Getting to know your instrument

The MPE 70 paraffin bath brings paraffin wax to melting point.

Dry and chapped hands, feet or elbows are submerged repeatedly for short periods of time into the warm liquid paraffin wax and then removed again. This causes a thin layer of wax to form around the affected part of the body, over which a sheet is additionally placed. The warmth of the paraffin wax opens the pores of the skin and moisture escapes. The layer of wax prevents the moisture from escaping. Instead, the moisture enters the opened pores and penetrates deep into the skin, where it is absorbed. This makes skin especially soft and smooth. After an exposure time of 20 minutes, the paraffin wax can be removed from the affected body parts.

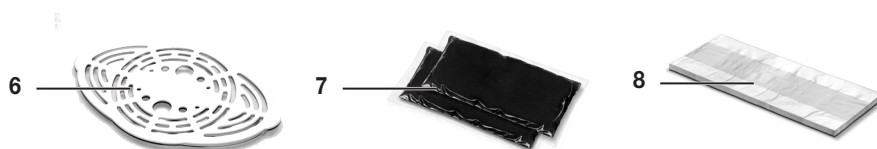
In addition, the warmth of the paraffin bath improves circulation.

4. Intended use

This device is intended exclusively for use on the hands, feet or elbows of humans. Do not use the device on animals!

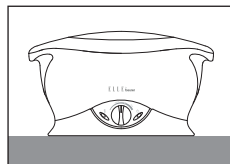
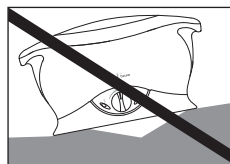
5. Unit description

1. Paraffin bath
2. Lid
3. Green operation light
4. Temperature controller
5. Red heating light
6. Protective grille
7. Paraffin wax
8. Plastic sleeves

6. Initial use

- Remove the packaging material.
- Make sure that all parts specified in the Included in delivery list are contained (see 1. Included in delivery).
- Check the paraffin bath and mains cable for damage.
- Place the paraffin bath on a flat, secure, even and water-resistant surface or table.
- Make sure that the paraffin bath is empty before filling it with paraffin wax.

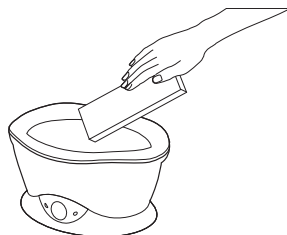
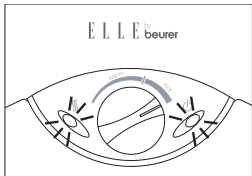
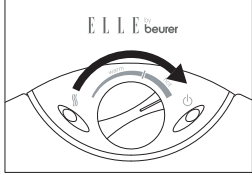
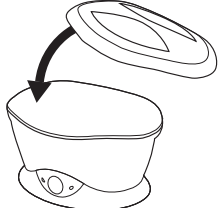
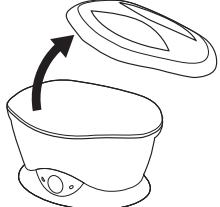
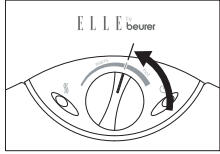
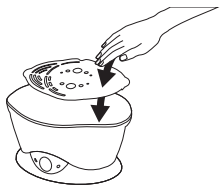
- Remove the paraffin wax from its plastic packaging.
- Put the desired amount of paraffin wax (minimum 0.9 kg, maximum 1.35 kg) into the paraffin bath. In order to make it easier to put the paraffin wax into the paraffin bath, break up the blocks of paraffin wax.

Important

Make sure that the paraffin bath is always filled with paraffin wax up to at least the MIN fill indicator line before use.

• Insert the mains plug into the socket (220-240V, ~50/60Hz). The green operation light on the paraffin bath will light up. The device is now switched on. The red heating light on the paraffin bath will also light up, and will only go out when the specified heat setting has been reached.	
• Turn the temperature controller to “hot” to melt the paraffin wax.	
• Now place the lid on the paraffin bath and wait until the paraffin wax has completely melted (around 2-3 hours). ⚠ Warning If the paraffin bath is set to “hot”, never dip your hands, feet or elbows in the paraffin wax.	
• As soon as the paraffin wax has completely melted, remove the lid. • Turn the temperature controller to the marked line between “warm” and “hot”. • Lay the protective grille included in the delivery contents in the liquid paraffin wax and wait until it has sunk to the bottom of the paraffin bath. • Put the lid back onto the paraffin bath and wait approximately one hour, until the paraffin wax has cooled to an ideal operating temperature (60°C). ⚠ Warning The room temperature can affect the power of the device and consequently the temperature of the wax. Before submerging your hands, feet or elbows in the paraffin bath, use the inside of your wrist to check that the temperature of the wax is comfortable. If a skin should form on the wax surface, increase the temperature by turning the temperature controller clockwise. If the paraffin wax is too hot for you, reduce the temperature by turning the temperature controller anti-clockwise.	  

7. Operation

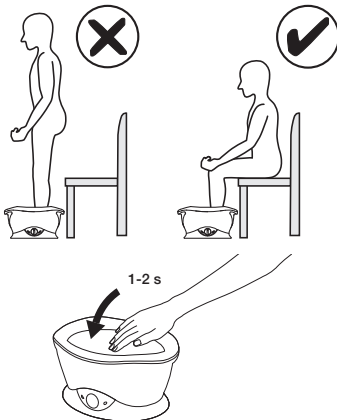
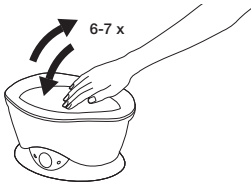
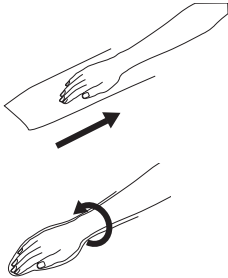
⚠ Warning

Before submerging your hands, feet or elbows in the paraffin bath, use the inside of your wrist to check that the temperature of the wax is comfortable.

Warning

Take off your jewellery (e.g. watches, rings, etc.) before treatment.

After you have completed the necessary preparations from “Chapter 6 Initial use”, you can begin using the device. To start the treatment, carry out the following steps:

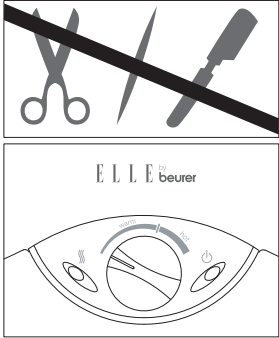
• We recommend applying a lanolin-based hand lotion to the areas to be treated before treatment. This will help you achieve a better result, and will make it easier to remove the paraffin wax after treatment.  Warning When using the paraffin bath on your feet, only use it sitting down. Never stand in the paraffin bath. • Briefly submerge the area to be treated (your hand, foot or elbow) for 1-2 seconds in the liquid paraffin wax. Remove your hand again and wait briefly until the wax has solidified. During this time, do not move the treated area, so that the wax can solidify properly.	
• Repeat the process of submerging and pulling out six to seven times, until several layers of paraffin wax have formed over the treatment areas. If the paraffin wax is either too hot or too cold for you during the treatment period, turn the temperature controller clockwise or anti-clockwise, to increase or decrease the temperature as required.	
• Now put a sheet on the treated area, press it onto the paraffin wax and twist the end shut. Keep the sheet on for 20 minutes.  Warning If you are wearing the plastic sleeves on your feet, do not run around in them. You could slip.	
• After 20 minutes, take the sheet and the layers of wax off the treated area. Dispose the sheet in the general waste; it cannot be reused.	
• After each use, take the protective grille out of the paraffin bath.	

8. Cleaning

Important

If the device is only being used by one person, change the paraffin wax at regular intervals and clean the paraffin bath whilst doing so.

If the device is being used by several people, the paraffin wax should be changed before each treatment for reasons of hygiene.

• Pull the mains plug out of the socket. • Take the protective grille out of the paraffin bath. • Allow the paraffin wax to solidify overnight.	
 Warning Do not use any sharp or pointed objects to remove the wax block from the paraffin bath. You could damage or scratch the paraffin bath. • If the paraffin wax has solidified, press on one end of the wax block and lift it out of the paraffin bath. If the wax block cannot be removed easily, turn the temperature controller to “warm” for ten minutes. Then remove the mains plug and take the wax block out of the paraffin bath.	
• Then wipe the inside of the paraffin bath dry with a clean towel.  Important Never fill the paraffin bath with water in order to clean it.	
• After cleaning, store the paraffin bath in a cool, dry and dust-proof place.	

9. Replacement parts and wearing parts

	Item and/or order number
2 x 450 g of paraffin wax + 30 plastic sleeves	589.42*

*Commercially available or from the corresponding service address (according to the service address list)

10. Disposal

Please dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment).

If you have any queries, please contact the appropriate local authorities.

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